NO•DOL

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NO•DOL

Medical devices with natural ingredients

CREAM

Protective cream that provides relief from joint pain and stiffness.

Recommended for its protective and soothing properties, it helps alleviate joint pain and stiffness.

How to use: apply the cream to the affected areas at least twice a day and massage in until absorbed.

Ingredients: Aqua, C12-20 Acid PEG-8 Ester, Dimethyl Sulfone, Glucosamine Sulfate, Caprylic/capric Triglyceride, Isopropyl Palmitate, Glycerin, Ethylhexyl Palmitate, Cetyl Alcohol, Glyceryl Stearate, PEG-100 Stearate, Petrolatum, Dimethicone, Prunus Amygdalus Dulcis Oil, Cyclopentasiloxane, Butyrospermum Parkii Butter, Propylene Glycol, Gluconolactone, Phenoxyethanol, Tocopheryl Acetate, Sodium Benzoate, Sodium Hydroxide, Aloe Barbadensis Leaf Juice, Calcium Gluconate, Potassium Sorbate, Harpagophytum Procumbens Root Extract, Salix Alba Bark Extract, Alcohol, Benzyl Alcohol, Crocus Sativus Extract, CI 19140, Arnica Montana Flower Extract, Citric Acid.

Warnings: Do not use this product if the packaging is open or damaged. Do not ingest the product. Do not use after the expiry date. Keep out of reach of children. Store in a cool place away from sources of heat. Close the container after use. The expiry date is valid only for unopened products that have been stored correctly. Avoid using on broken skin or around the eyes. Discontinue use if irritation or hypersensitivity to the product occurs and consult your doctor if symptoms persist. This product has not been tested on pregnant women or women who are breastfeeding.

Contraindications: Contraindicated in patients suffering from hypersensitivity to any of the ingredients. 100 ml tube

MEDICAL DEVICE

Symbol	Meaning	Symbol	Meaning
	Medical device manufacturer		Store away from sources of heat at a temperature between 5° and 30° C
	CE marking		Expiry date
	Read the warnings and instructions for use		Lot number

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